

# Replens Long Lasting Vaginal Moisturiser



## Category

Female Health and  
Genitourinary

## Manufacturer

CHURCH & DWIGHT UK  
LTD

## Pack size

35g

## RRP

£11.55

## Legal Status

MD



## Uses

Long lasting vaginal moisturiser. Helps replenish vaginal moisture and alleviates vaginal dryness symptoms. One application provides up to 3 days' relief. Compatible with most condoms.



## Format and Ingredients

Tube and re-usable applicator



## Dosage

Adults: remove cap from tube and break seal by puncturing with the opposite end of the cap. Screw the open end of the applicator onto the tube and pull the end of the applicator gently until it comes to a natural halt (approximately 4cm from the end of barrel). Fill the empty part of the barrel with approximately 4cm of gel. While sitting, standing or lying on your back with knees bent, gently insert open end of applicator into the vagina as deeply as it will go comfortably. Hold the applicator in place with thumb and middle finger and press the plunger until it stops. Withdraw the applicator. After use, pull the plunger all the way out of the barrel. Wash both parts of the applicator in warm soapy water. Rinse through thoroughly and dry. To reassemble, gently push the plunger back into the barrel as far as it will go.